


Four Appointment Denture Workflow

	Appointment 1 Impression & Preliminary Bite	Appointment 2 Bite Registration	Appointment 3 Wax Try In	Appointment 4 Delivery
Receive from DPS	N/A	<ul style="list-style-type: none"> • Maxillary master model • Mandibular master model • Bite block (BB) 	<ul style="list-style-type: none"> • Maxillary master model • Mandibular master model • Wax set up for try in 	<ul style="list-style-type: none"> • All models • All bites • Finished denture
In-Office Steps	<ul style="list-style-type: none"> • Take maxillary and mandibular full arch impression with border molding, using a medium body PVS or alginate • Take preliminary full arch bite or mush bite for edentulous 	<ul style="list-style-type: none"> • Verify vertical dimension • Take full arch bite • Verify midline and cuspid placement on BB • Mark high lip line on BB • Mark low lip line on BB • Select the tooth shade and mould 	<ul style="list-style-type: none"> • Verify vertical dimension • Verify bite • Evaluate esthetics and speech 	<ul style="list-style-type: none"> • Seat the denture • Verify vertical • Verify bite • Verify esthetics and speech • Adjust denture as needed
Tips	If using alginate, pour the impression immediately.	Trim bite block if needed, cover entire occlusal of bite block with bite material and guide the patient to close into centric.	If the bite is off, take a new full arch bite with the denture seated. Check for "F" sounds and "S" sounds.	Counsel patients to avoid cleaners containing peroxide or bleach.
Send to DPS	<ul style="list-style-type: none"> • Maxillary impression/model • Mandibular impression/model • Preliminary full arch bite • Pre-op model (if available) • Rx 	<ul style="list-style-type: none"> • Maxillary master model • Mandibular master model • Bite block • Full arch bite • Tooth shade and mould • Pre-op model (if available) • Rx #2 	<ul style="list-style-type: none"> • Maxillary master model • Mandibular master model • Wax set up • Any new bites • Full arch bite • Additional information or instructions for changes* • Rx #3 	
DPS Steps	<ul style="list-style-type: none"> • Create maxillary master model • Create mandibular master model • Create bite block (BB) • Articulate to preliminary bite 	<ul style="list-style-type: none"> • Articulate to new bite • Create wax set up for try in 	<ul style="list-style-type: none"> • Verify set up • Process denture • Mill in bite • Finish denture 	
Number of Business Days Required for Production <i>(Does not include pick up or delivery days)</i>	2 days in lab	2 days in lab	3 days in lab	



DENTAL PROSTHETIC SERVICES
 1900 51st Street NE, Cedar Rapids, Iowa 52402
 800-332-3341 Fax: 319-3938455
 www.DPSdental.com

**If an additional try in is needed, repeat Appointment 3.*