


Four Appointment Cast Frame Partial Workflow

	Design & Estimate Optional	Appointment 1 Impression & Preliminary Bite	Appointment 2 Framework Bite Registration	Appointment 3 Wax Try In	Appointment 4 Delivery
Receive from DPS		<ul style="list-style-type: none"> Design layout Estimate Custom trays (optional) 	<ul style="list-style-type: none"> Maxillary master model Mandibular master model Frame with BB 	<ul style="list-style-type: none"> Maxillary master model Mandibular master model Wax set up for try in 	<ul style="list-style-type: none"> All models All bites Completed cast frame partial
In-Office Steps	<ul style="list-style-type: none"> Take maxillary and mandibular full arch impression with border molding, using a medium body PVS or alginate Take bite 	<ul style="list-style-type: none"> Take maxillary and mandibular full arch impression with bordering molding, using a medium body PVS or alginate Take preliminary full arch bite 	<ul style="list-style-type: none"> Verify vertical dimension Take full arch bite Verify fit of framework Mark midline on BB Mark high lip line on BB Mark low lip line on BB Select the tooth shade and mould 	<ul style="list-style-type: none"> Verify vertical dimension Verify bite Evaluate esthetics and speech 	<ul style="list-style-type: none"> Seat the cast frame partial Verify vertical Verify bite Verify esthetics and speech Adjust acrylic on cast frame partial, if needed
Tips	This is a complimentary service to help in case planning	If using alginate, pour the impression immediately.	Trim bite block if needed, cover entire occlusal of bite block with bite material and have the patient close into centric. If metal is occluding, take photos in the mouth, and email to DPS at shades@dpsdental.com	If the bite is off, take a new full arch bite with the cast frame partial seated. Check for "F" sounds and "S" sounds.	Counsel patients to avoid cleaners containing peroxide, chlorine, or bleach.
Send to DPS	<ul style="list-style-type: none"> Study model Opposing arch model Bite Rx 	<ul style="list-style-type: none"> Maxillary impression/model Mandibular impression/model Preliminary full arch bite Pre-op model (if available) Design and estimate (or note it on the Rx) Rx #2 	<ul style="list-style-type: none"> Maxillary master model Mandibular master model Frame with bite block Full arch bite Tooth shade and mould Pre-op model (if available) Rx #3 	<ul style="list-style-type: none"> Maxillary master model Mandibular master model Wax set up Any new bites Full arch bite Additional information or instructions for changes* Rx #4 	
DPS Steps	<ul style="list-style-type: none"> Create a design and estimate 	<ul style="list-style-type: none"> Create maxillary master model Create mandibular master model Design and cast partial frame Finish and place bite blocks (BB) 	<ul style="list-style-type: none"> Articulate case Create wax set up for try in 	<ul style="list-style-type: none"> Verify bite Process and finish partial 	
Number of Business Days Required for Production <i>(Does not include pick up or delivery days)</i>	1 day in lab (D & E only) 2 days in lab (D & E and custom trays)	4 days in lab	2 days in lab	3 days in lab	

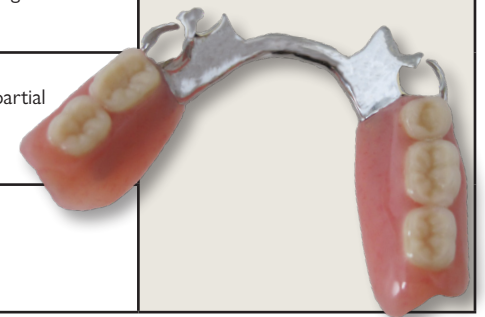

**If an additional try in is needed, repeat Appointment 3.*



DENTAL PROSTHETIC SERVICES
1900 51st Street NE, Cedar Rapids, Iowa 52402
800-332-3341 Fax: 319-3938455
www.DPSdental.com